



XPress Lock™

Reusable Monopolar Instruments

XPress Lock™ Line

Key Features




- Its innovative **quick-lock mechanism** allows for assembly and disassembly within seconds
- **Fully intuitive** and reliable
- **More than 120 different instrument patterns**, from multiple graspers, dissectors, specialty instruments and Ceramix Cut™, a high performance Metzenbaum scissor insert featuring ceramic coated blades
- Also available in **Ø 3 mm for Miniature Laparoscopy** procedures

How Quick-Lock works

Disassembly of XPress Lock™ instruments

- 1 Open the handle completely.
- 2 Loosen the sheath by pressing the quick-locking knob on the black screw connector.
- 3 While holding the sheath, rotate the insert clockwise 45° until loosened. Slide the insert from sheath.

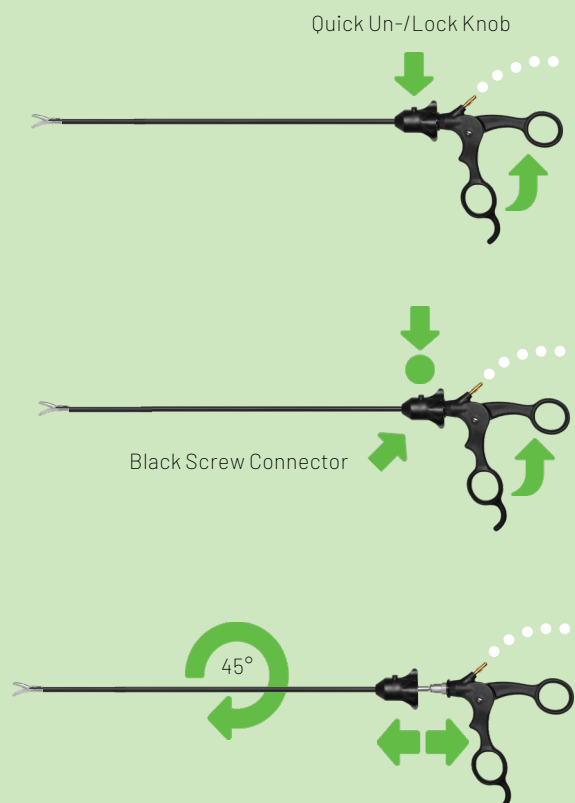



Ceramix Cut Scissors

Key Features and more

- Wear-resistant Metzenbaum scissors
 - Suitable for use with XPress Lock™
 - Outstanding cutting ability
 - Reduced abrasion
 - Minimal optical reflection
 - Substantially harder than regular stainless steel
-
- Cost-effective due to long-term stable surface
 - Solid and non-corrosive
 - Temperature- and scratch-resistant
 - Smooth and dense surface and easily cleanable
 - More resistant to detergents and disinfectants
 - Improved sliding characteristics, therefore smoother movement of the scissor blades



XPress Lock™ Topsellers

Ordering Information and Recommendations

1x Metzenbaum scissors, curved, ceramic, Ø 5 mm, 330 mm <i>(insert)</i>	# 13-1461CI
1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm	# 13-1356XP
1x XPress Lock™ handle w/o ratchet	# 13-1367XP
1x Hook scissors, Ø 5 mm, 330 mm <i>(insert)</i>	# 13-1431XPI
1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm	# 13-1356XP
1x XPress Lock™ handle w/o ratchet	# 13-1367XP
1x Maryland dissector, Ø 5 mm, 330 mm <i>(insert)</i>	# 13-1318XPI
1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm	# 13-1356XP
1x XPress Lock™ handle w/o ratchet	# 13-1367XP
1x Endoscopic clinch, atraumatic, Ø 5 mm, 330 mm <i>(insert)</i>	# 13-1323XPI
1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm	# 13-1356XP
1x XPress Lock™ handle with universal ratchet	# 13-1368XP
1x Standard grasper, cross-serrated, Ø 5 mm, 330 mm <i>(insert)</i>	# 13-1354XPI
1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm	# 13-1356XP
1x XPress Lock™ handle with universal ratchet	# 13-1368XP
1x DeBakey grasper, Ø 5 mm, 330 mm <i>(insert)</i>	# 13-1570XPI
1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm	# 13-1356XP
1x XPress Lock™ handle with universal ratchet	# 13-1368XP
1x Johan grasper, slightly flat jaws, Ø 5 mm, 330 mm <i>(insert)</i>	# 13-1312XPI
1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm	# 13-1356XP
1x XPress Lock™ handle with universal ratchet	# 13-1368XP



- 1x Babcock grasper, curved, ceramic, Ø 5 mm, 330 mm (*insert*)
- 1x Sheath for XPress Lock™ inserts, Ø 5 mm, 330 mm
- 1x XPress Lock™ handle with universal ratchet

13-1541XPI
13-1356XP
13-1368XP



- 1x Claw forceps, traumatic jaws, Ø 10 mm, 330 mm (*insert*)
- 1x Sheath for XPress Lock™ inserts, Ø 10 mm, 330 mm
- 1x XPress Lock™ handle with universal ratchet

13-1605XPI
13-1357XP
13-1368XP



- 1x Biopsy spoon forceps, Ø 10 mm, 330 mm (*insert*)
- 1x Sheath for XPress Lock™ inserts, Ø 10 mm, 330 mm
- 1x XPress Lock™ handle w/o ratchet

13-1660XPI
13-1357XP
13-1367XP





www.ackermannsurgical.com



Ackermann Instrumente GmbH
Eisenbahnstrasse 65-67
78604 Rietheim-Weilheim, Germany

Revision 01/23